


15800 NW 15th Avenue, Miami, FL 33169 **TOLL-FREE:** (877) QHC-BINS / (877) 742-2467 **FAX:** (877) 742-2497 **E-MAIL:** sales@ghcdirect.com

www.qhcdirect.com

HAMPER STAND WITH FOOT PEDAL ACTIVATED LID

Your carton should contain the following items.

NO.	DESCRIPTION	QTY
1	Lid	1
2	Tie Rod	1
3	10mm Nut	6
4	Connecting Rod	4
(5)	Adjusting Nut	2
6	Uprights	2
7	Base	1
8	Caster	2
9	Locking Caster	2
10	Wrench10×14mm	1

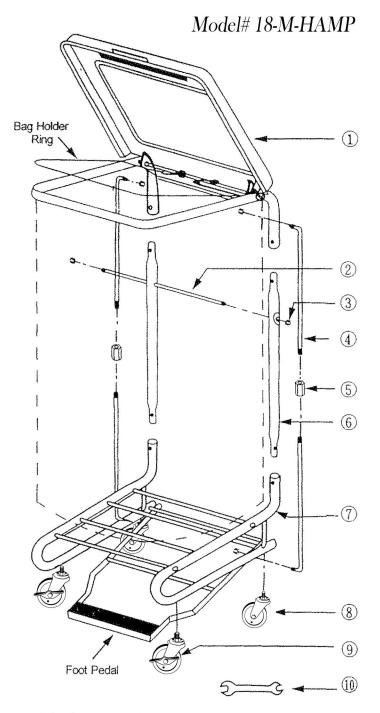

Listed below are your easy to follow assembly instructions:

- A. Attach Casters (a) and (a) to the Base with Lock Washers provided. Make sure Locking Casters (a) are placed in the front.
- B Insert Uprights (6) into Base ends (7) (allow push buttons to snap into position).
- C Insert Tie Rod ② into holes in Uprights ⑥ and attach with 10mm Nuts ③.
- D. Attach Lid ① to upper ends of Uprights ⑥ (allow push buttons to snap into position).
- E. Attach Connecting Rods (4) with Adjusting Nuts (5)
 Attach Connecting Rods (4) to lid and foot
 pedal at both sides of hamper with 10mm Nuts (3)
 provided. Connecting Rods ends should face
 toward inside of stand.